 made
in
Germany

Since 1948

 Thalheimer

Forensic

Forensic

Thalheimer Kühlung GmbH & Co. Kg. Was established in Germany in 1948. Initially the company specialized in Medical Cold/ Refrigerated equipment and utilities. Having thrived in this field it moved on to produce well- designed custom made various products that were well received in German and European markets. The company progressively grew in diversified outputs and ascending revenues. The company's acclaimed reputation reached the Middle East, Africa and Central Asia.

In appreciation of its reputation and craftsmanship, Thalheimer was elicited in 2009 to join the **ThalSa Group** stationed in Dubai, U.A.E.

Having joined the group, Thalheimer propelled to broader international fame and reputation. In its present status, the company is expanding its facilities to meet an expected increased demand for its products and to enjoy sustained improvement in quality, productivity and export.

Member of Thalsa Group

Forword

Our Mission

Behind every achievement there is a human touch and talent.

Searching for the missing link then proceed.

Continued improvement.

Diversification & innovation.

Encouraging multi-disciplinary team work.

Commitments stems from:

- Self confidence
- Adopting best technology
- Adherence to standards and strict quality control
- Shoulder responsibility, flexibility and efficency

Thalheimer Kühlung was established in 1948 to manufacture various cooling equipments, cold rooms, refrigerators and ice makers for brewery and catering business. Having succeeded in that field, our company got engaged in producing new products ever since.

In the 1960s Thalheimer introduced the first medical cooling equipment that was designed and added to the product's portfolio. Having joined Thalheimer Kühlung in the late 1960s we soon decided to direct the company towards the healthcare environment rather than the popular air conditioning that relates more to comfort and less to the health of people. At that time the development of cold hygienic blood bank refrigerators was being given top priority since donated blood had to be looked after in two crucial ways: safe way of storing and the best method of dispensing.

In the 1970s we developed refrigerators for the use in laboratory, pharmaceutical and pathology fields subsequently Thalheimer was in a capable position to fulfill all refrigeration requirements for Medical use in hospitals, universities and research institutions. The growing experience in manufacturing cooling systems allowed Thalheimer to develop ultra-low temperature freezers (down to -90°C) in addition to all requirements of industrial, commercial cold rooms and refrigerated warehouses.

We are also proud of our renown excellence in the design, manufacture and supply of state-of-the-art forensic requirements including mortuaries of various types and sizes. We have designed and offered complete solution for forensic establishment that full

filled the particular requirement and specifications of different clients in Germany and Europe.

Each product incorporates the latest technology, German precise workmanship and cost effectiveness. All built-in materials and components are of the highest quality. As a result, our equipment have the lowest noise level, easy to clean and offer hygienic satisfaction. Our ability to satisfy all demands of our customers regarding color, size, surface materials and modes of servicing indicates our particular strength.

We do not neglect special requirements; we find answers. Our company can provide you not only with a complete Medical refrigeration solution system but also with the constant supply of spare parts, installation and maintenance service and training courses.

The moto of our company is (conviction precedes persuasion); thus we are convinced of the excellent quality of our work, and aspire to see this conviction shared with our customers.

We hope that this brochure will convince you of our genuine workmanship and look forward to be of service to you.

Eng. Franz Ullmann
Chief Technical officer

Contents

Mortuary Refrigeration 6 - 37

- General Description 10
- Trays and Cooling System 12
- Technical Data 19
- Two cells Mortuary Refrigeration 22
- Two cells Mortuary Refrigerator / Side loading 24
- Four cells Mortuary Refrigerator 26
- Six cells Mortuary Refrigerator 28
- Six cells Mortuary Refrigerator / Tow doors 30
- Nine cells Mortuary Refrigerator 32
- Twelve cells Mortuary Refrigerator 36

Transport carts 38- 49

- Lifting and Transport Cart LTW-MHY 42

Dissection Tables 50 - 59

- Autopsy table KST for anatomical classes 52
- Washing and Autopsy Table WT 58

Thalheimer mortuary refrigerators LKZ CC / LKZT CC are produced with emphasis on modern Technology, economic efficiency and German preciseness.

Thalheimer mortuary refrigerators can be assembled variably by means of cam-locks system, that Guarantees stability and tightness and makes possible later extension or dismounting for moving.

Mortuary Refrigeration

Mortuary Refrigerators

Preservation temperature:	+4 °C (tolerance ±1.5 °C) digital indication
Ambient temperature:	up to +32 °C / +50 °C on request
Exterior material:	Sheet steel lacquered white OR stainless steel type 1.4301 (AISI 304)
Interior material:	Sheet steel lacquered white OR stainless steel type 1.4301 (AISI 304)
Insulation:	Polyurethane, CFC-free, high density foam 80 mm thick, heat transfer coefficient 0,240 W/m² K
Doors:	Variable construction, door frame heating, locks can be opened from inside by pushing (Push-In), cell door's dimensions (W/H) 680 x 500 mm (standard), other doors (large vertical or horizontal doors) on request
Roller track:	One roller track per body, with 5 rolls, covered with plastic, parts easy removable and cleanable
Body tray:	Stainless steel seamless deep-drawn, with 2 handles
Cooling system:	Can be mounted on the top panel, alongside the refrigerator or remotely, hermetically sealed, air-ventilated, vibration-free, refrigerant R134a or R404a CFC-free, automatic time controlled defrosting, safety thermostat, microprocessor controller
Alarm system:	Visual and acoustic signaling at temperature instability, voltage 230 V or batteries, can be connected to remote alarm control through potential-free terminal
Voltage:	230V/50Hz/1ph other voltage on request
Condensation water drainage:	Drainage piping from evaporator to the floor drain through bottom panel or side panel is possible, interior evaporation of condensation water is also possible
Optional accessories:	Interior light with separate switch or door contact-switch, Door frame heating, Label holder on the doors, Blinds for ceiling and sides, Temperature recorder 7 days / 24 hours, with batteries, Water-cooled refrigeration system, Humidity control, Additional cooling system automatically switched on if failure occurs

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Mortuary Freezers

Preservation temperature:	-20 °C (tolerance ±1,5 °C) digital indication
Ambient temperature:	up to +32 °C / +50 °C on request
Exterior material:	Sheet steel lacquered white OR stainless steel type 1.4301 (AISI 304)
Interior material:	Sheet steel lacquered white OR stainless steel type 1.4301 (AISI 304)
Insulation:	Polyurethane, CFC-free, high density foam 100 mm thick, heat transfer coefficient 0,20 W/m² K
Doors:	Variable construction, door frame heating, locks can be opened from inside by pushing (Push-In), cell door's dimensions (W/H) 680 x 500 mm (standard), other doors (large vertical or horizontal doors) on request
Roller track:	One roller track per body, with 5 rolls, covered with plastic, parts easy removable and cleanable
Body tray:	Stainless steel seamless deep-drawn, with 2 handles
Cooling system:	Can be mounted on the top panel, alongside the refrigerator or remotely, hermetically sealed, air-ventilated, vibration-free, refrigerant R507 CFC-free, automatic time controlled defrosting, safety thermostat, microprocessor controller
Alarm system:	Visual and acoustic signaling at temperature instability, voltage 230 V or batteries, can be connected to remote alarm control through potential-free terminal
Voltage:	230V/50Hz/1ph other voltage on request
Condensation water drainage:	Drainage piping from evaporator to the floor drain through bottom panel or side panel is possible, interior evaporation of condensation water is also possible
Optional accessories:	Interior light with separate switch or door contact-switch, Label holder on the doors, Blinds for ceiling and sides, Temperature recorder 7 days / 24 hours, with batteries, Water-cooled refrigeration system, Humidity control, Additional cooling system automatically switched on if failure occurs

Our mortuary freezers have the lowest noise level, are easy to clean and meet hygienic requirements better.

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When the machine is built on top of refrigerator the total height = exterior height + machine screen

Capacity	Arrangement				Screen	Conn. load		
Type	Bodies	Vertical	Rows	Width, mm	Depth, mm	Height, mm	VA	
CC1	1	1	1	1250	2300	1250	250	1250
CCN2	2	2	1	1250	2300	1850	350	1250
CC3	3	3	1	1250	2300	2300	350	1250
CCN4	4	2	2	1950	2300	1850	350	1400
CCN6	6	3	2	3000	2300	1850	350	1400
CC6	6	3	2	1950	2300	2300	350	1400
CC9	9	3	3	3000	2300	2300	350	1400
CC12	12	3	4	3700	2300	2300	350	2400
CC15	15	3	5	4750	2300	2300	350	2400

When the machine is built on top of refrigerator the total height = exterior height + machine screen

General Description

General Description

Thalheimer mortuary or coffin refrigerators have modular construction of sandwich panel system. The assembly of the cells is simple with a groove and a tongue; foamed-in eccentric hook locks of the panels guarantee stability and tightness, a later extension or dismantling for moving is possible. The sandwich panels consist of two layers of sheet steel covered with plastic or stainless steel filled with foamed-in polyurethane CFC-free. Polyurethane with thickness of 80 mm has heat transfer coefficient 0,240W/m² (DIN 52612) which guarantees low energy consumption.

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Housing

Exterior and interior housings are built of white lacquered (RAL 9002) sheet steel 0,8mm or stainless steel, material 1.4301 (AISI 304), grinding finish grain 240. Different combinations such as lacquered sheet steel for the exterior and stainless steel for interior or vice versa are possible. The floor, wall and ceiling panels are assembled together with a groove, a tongue and foamed-in eccentric hook locks, an absolute sure and stable construction.

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Doors

The refrigerator can be equipped with a single door for one compartment or with a bigger door for several compartments. Door frame heating is built in to prevent water condensation. The heavy duty hinges are made out of hard chrome; the door can be locked from outside, but always it can be open from inside by pushing the door. If the preservation temperature of a refrigerator is below zero, the doors are provided with pressure compensation valves.

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Trays and Cooling System

Tracks and Trays

One roller track with 5 rolls ø 45mm with maintenance-free ball bearings for easy movement in and out of the tray is provided for each cell, at the rear part it has an additional stop-roll which prevents against hitting the back wall. The rolls are made of corrosion resistant material and have maintenance free ball bearings; the rails are completely made of stainless steel. The roller track is removable without tools. The stainless steel body tray, material 1.4301 (AISI 304) is seamless deep-drawn and has two handles.

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Cooling System

Cooling system can be mounted on the top of the refrigerator, alongside or remotely outside the building. The machine is hermetically sealed, airventilated and vibration free. Refrigerant is R134a or R404a, CFC-free. High performance evaporator is installed inside with automatic time-controlled defrosting and safety thermostat, microprocessor controlled. Alarm starts for too high / too low temperature, voltage 230V or with accumulator in case the power fails. The alarm signal can be transferred to the remote alarm through a potential free terminal.

The cooling system can be mounted on a ceiling or side panel, remotely mounted machines are possible. Voltage is 230V, 50Hz, 1ph, other voltages on request. Condensation water drainage (heated in case of deep freezing), with pipes from the evaporator to the floor drain (optionally), interior evaporation of condensation water inside is also possible.

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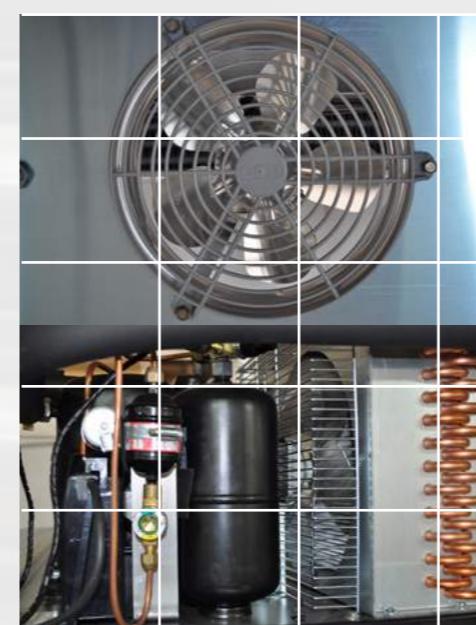
**made
in
Germany**

Preservation temperature

Refrigeration at +4° C and freezing at -20° C (tolerance $\pm 1.5^{\circ}$ C) digital indication, at ambient temperatures of +32° C.

Cooling systems for up to +50°C ambient temperature on request.

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Optional Accessories

- Interior light with a normal switch or a door contact switch
- Label holder on the doors
- Blinds for ceiling and sides
- Temperature recorder 7 days / 24 hours, battery operated
- Water cooled machines
- Additional cooling system automatically switches on if failure occurs

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Preparation at site

- Power supply 230V, 16A, 50 Hz, 1ph (plug box)
- Floor drainage with a siphon to prevent offensive smell

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Mortuary Refrigeration System

Mortuary Refrigerator LKZ / Mortuary Freezer LKZT

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Our mortuary refrigerators are equipped with

- time controlled defrosting,
- safety thermostat,
- microprocessor controller,
- alarm system which can be connected to remote alarm control system and
- other features that make the use of the mortuary refrigerator convenient and efficient.

Our mortuary refrigerators have the lowest noise level, are easy to clean and meet hygienic requirements better. Our ability to satisfy any wish of our customers regarding size, surface materials and way of serving equipment indicates our strength in particular. We do not produce mass-products - we find answers and solutions.

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Technical Data

Preservation temperature:	+4° (tolerance ±1.5° C) digital indication
Ambient temperature:	up to +32° C/ +50°C on request
Exterior material:	Sheet steel lacquered white OR stainless steel type 1.4301 (AISI 304)
Interior material	Sheet steel lacquered white OR stainless steel type 1.4301 (AISI 304)
Insulation:	Polyurethane, CFC-free, high density foam 80 mm thick, heat transfer coefficient 0,240 W/m² K
Roller track:	One roller track per body, with 5 rolls, covered with plastic, parts easy removable and cleanable
Body tray:	Stainless steel seamless deep-drawn, with 2 handles
Cooling system:	Can be mounted on the top panel, alongside the refrigerator or remotely, hermetically sealed, air-ventilated, vibration-free, refrigerant R134a or R404a CFC-free, automatic time controlled defrosting, safety thermostat, microprocessor controller
Alarm system:	Visual and acoustic signaling at temperature instability, voltage 230 V or batteries, can be connected to remote alarm control through potential-free terminal
Voltage:	230V/50Hz/1ph other voltage on request
Condensation water drainage:	Drainage piping from evaporator to the floor drain through bottom panel or side panel is possible, interior evaporation of condensation water is also possible

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A COMMITMENT DEMONSTRATED BY DISTINCTIVE VALUE


A global search for quality and environmental care: Thalheimer has achieved the **ISO 9001 certification** which has minimised its environmental impact in every aspect of production, from the supply of raw materials to manufacturing, from packaging materials to logistics. Further evidence of the ability to understand the market and interpret current trends and the changes taking place, asserting itself as a modern and dynamic organisation that can stand out from the competition.

Two cells Mortuary Refrigeration

Mortuary refrigerators keep +4° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).
 Mortuary freezers keep -20° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).
 Exterior and interior material can be sheet steel lacquered white or stainless steel on request.
 Polyurethane foam with low heat transfer coefficient 0.240 W/m² K is used as insulation.

Our mortuary refrigerators have the lowest noise level, are easy to clean and meet hygienic requirements better.
 Our ability to satisfy any wish of our customers regarding size, surface materials and way of serving equipment indicates our strength in particular.
 We do not produce mass-products - we find answers and solutions.

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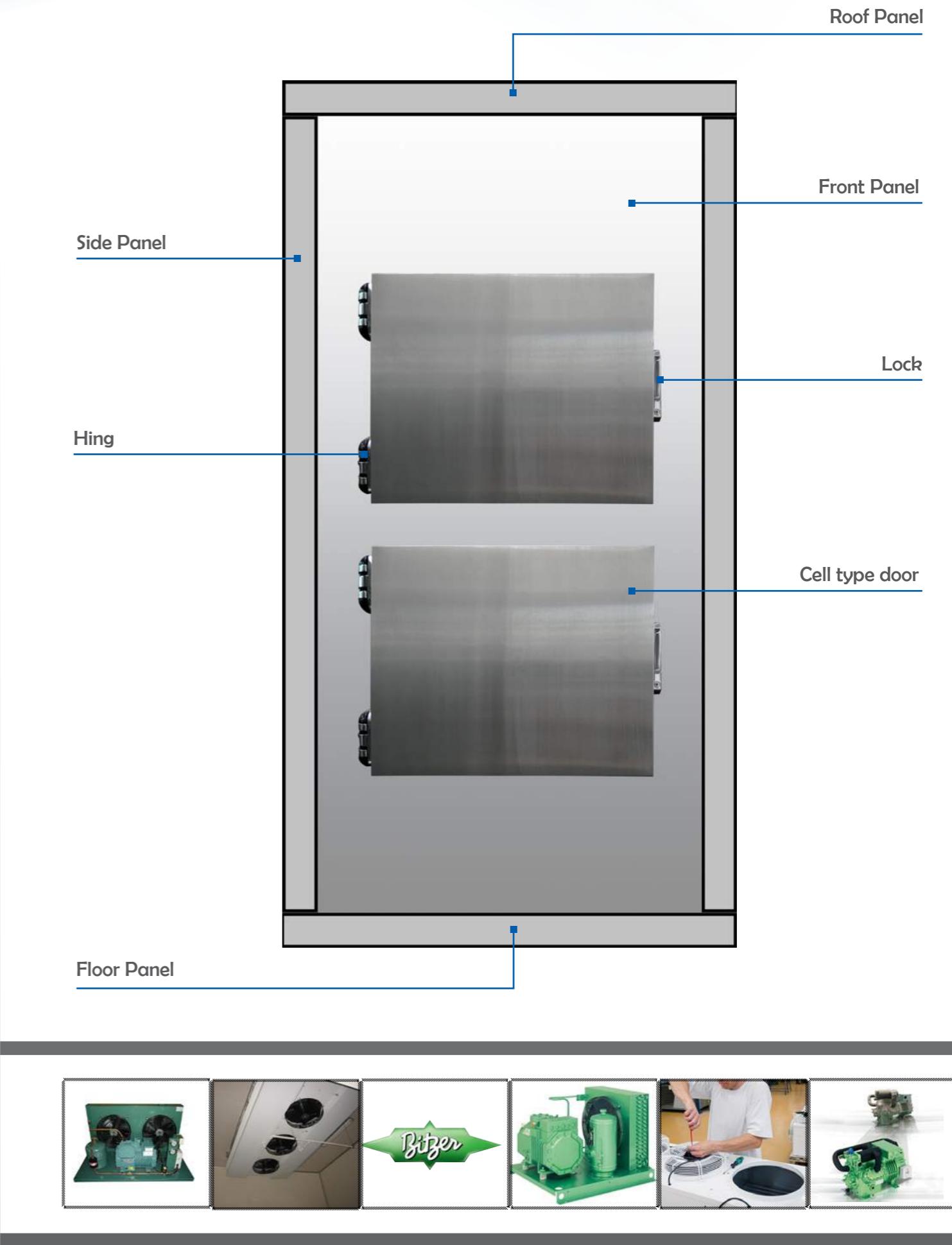
Three cells Mortuary Refrigeration

Mortuary refrigerators keep +4° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).
 Mortuary freezers keep -20° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).
 Exterior and interior material can be sheet steel lacquered white or stainless steel on request.
 Polyurethane foam with low heat transfer coefficient 0.240 W/m² K is used as insulation.

Our mortuary refrigerators have the lowest noise level, are easy to clean and meet hygienic requirements better.
 Our ability to satisfy any wish of our customers regarding size, surface materials and way of serving equipment indicates our strength in particular.
 We do not produce mass-products - we find answers and solutions.

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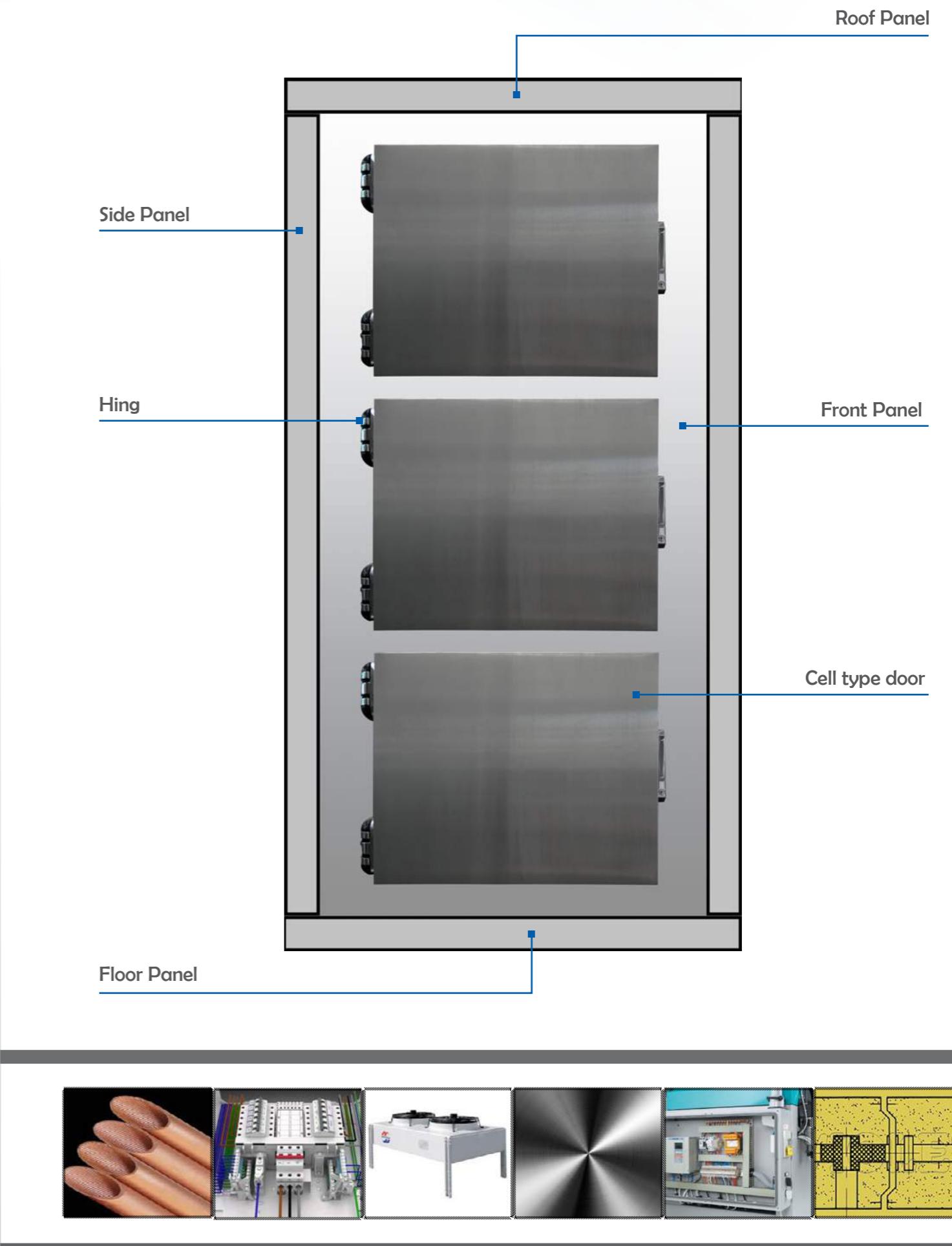Two cells Mortuary Refrigerator / Side loading

Mortuary refrigerators keep +4° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).

Mortuary freezers keep -20° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).

Exterior and interior material can be sheet steel lacquered white or stainless steel on request.

Polyurethane foam with low heat transfer coefficient 0.240 W/m² K is used as insulation.

Our mortuary refrigerators have the lowest noise level, are easy to clean and meet hygienic requirements better.

Our ability to satisfy any wish of our customers regarding size, surface materials and way of serving equipment indicates our strength in particular.

We do not produce mass-products - we find answers and solutions.

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
Four cells Mortuary Refrigerator

Mortuary refrigerators keep +4° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).
 Mortuary freezers keep -20° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).
 Exterior and interior material can be sheet steel lacquered white or stainless steel on request.
 Polyurethane foam with low heat transfer coefficient 0.240 W/m2 K is used as insulation.

Our mortuary refrigerators have the lowest noise level, are easy to clean and meet hygienic requirements better.
 Our ability to satisfy any wish of our customers regarding size, surface materials and way of serving equipment indicates our strength in particular.
 We do not produce mass-products - we find answers and solutions.

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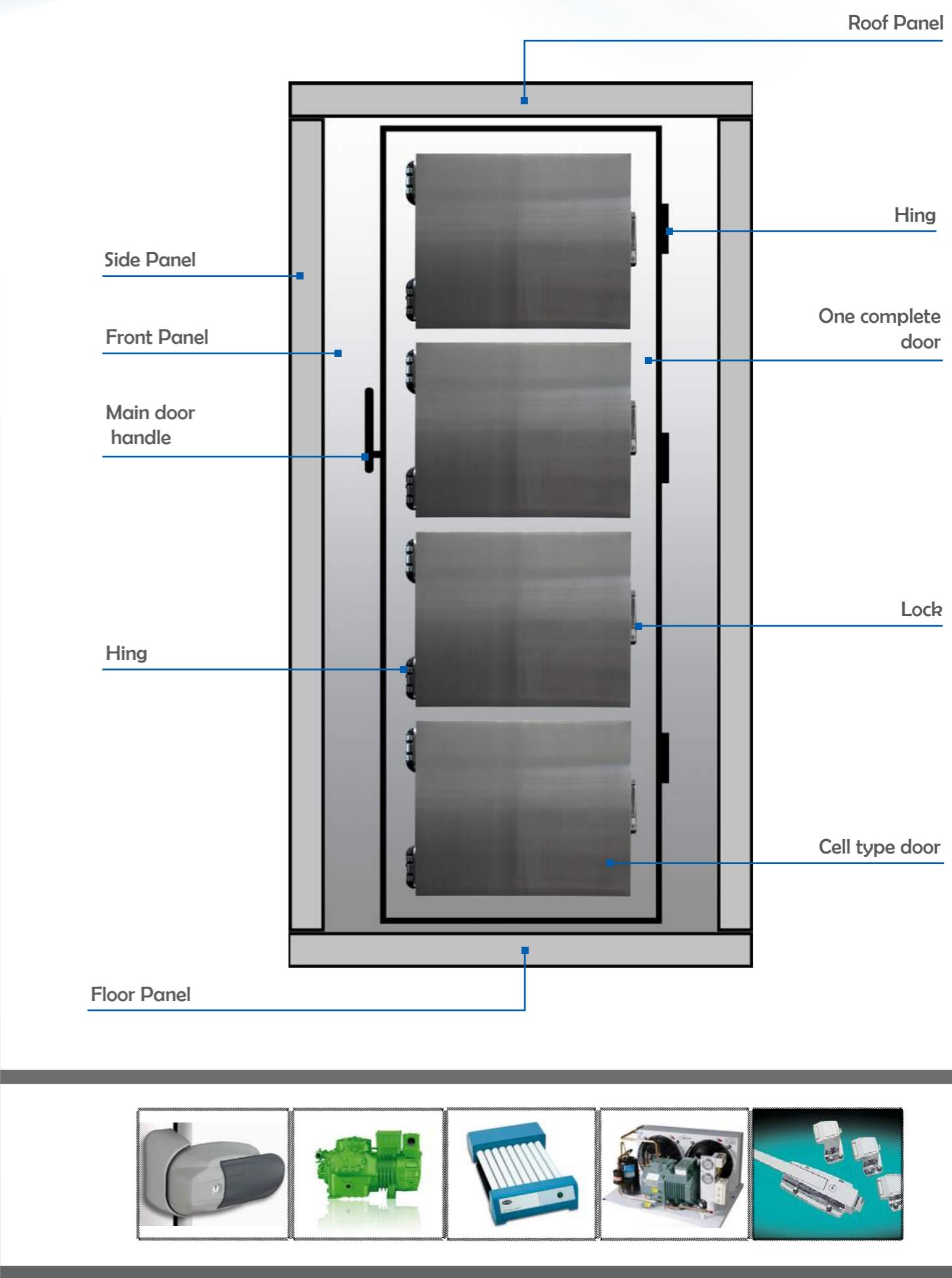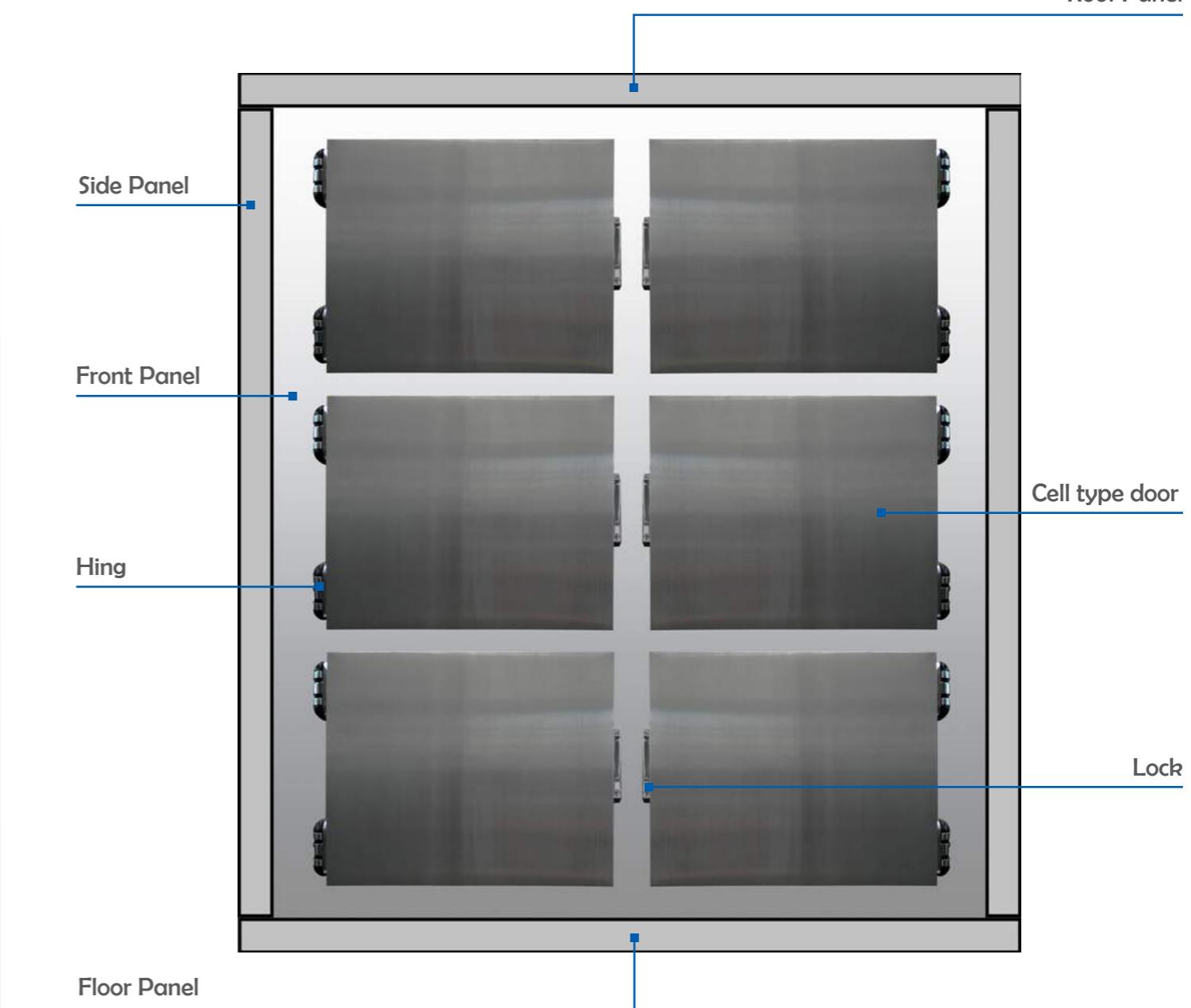
Six cells Mortuary Refrigerator / Tow doors

Mortuary refrigerators keep +4° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).

Mortuary freezers keep -20° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).

Exterior and interior material can be sheet steel lacquered white or stainless steel on request.

Polyurethane foam with low heat transfer coefficient 0.240 W/m² K is used as insulation.

Our mortuary refrigerators have the lowest noise level, are easy to clean and meet hygienic requirements better.



Our ability to satisfy any wish of our customers regarding size, surface materials and way of serving equipment indicates our strength in particular.

We do not produce mass-products - we find answers and solutions.

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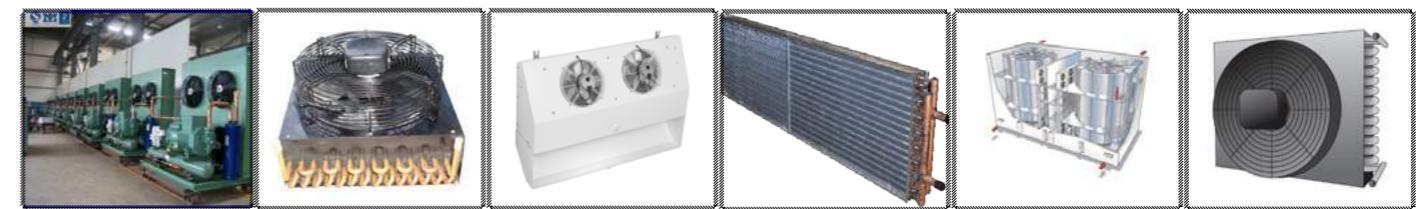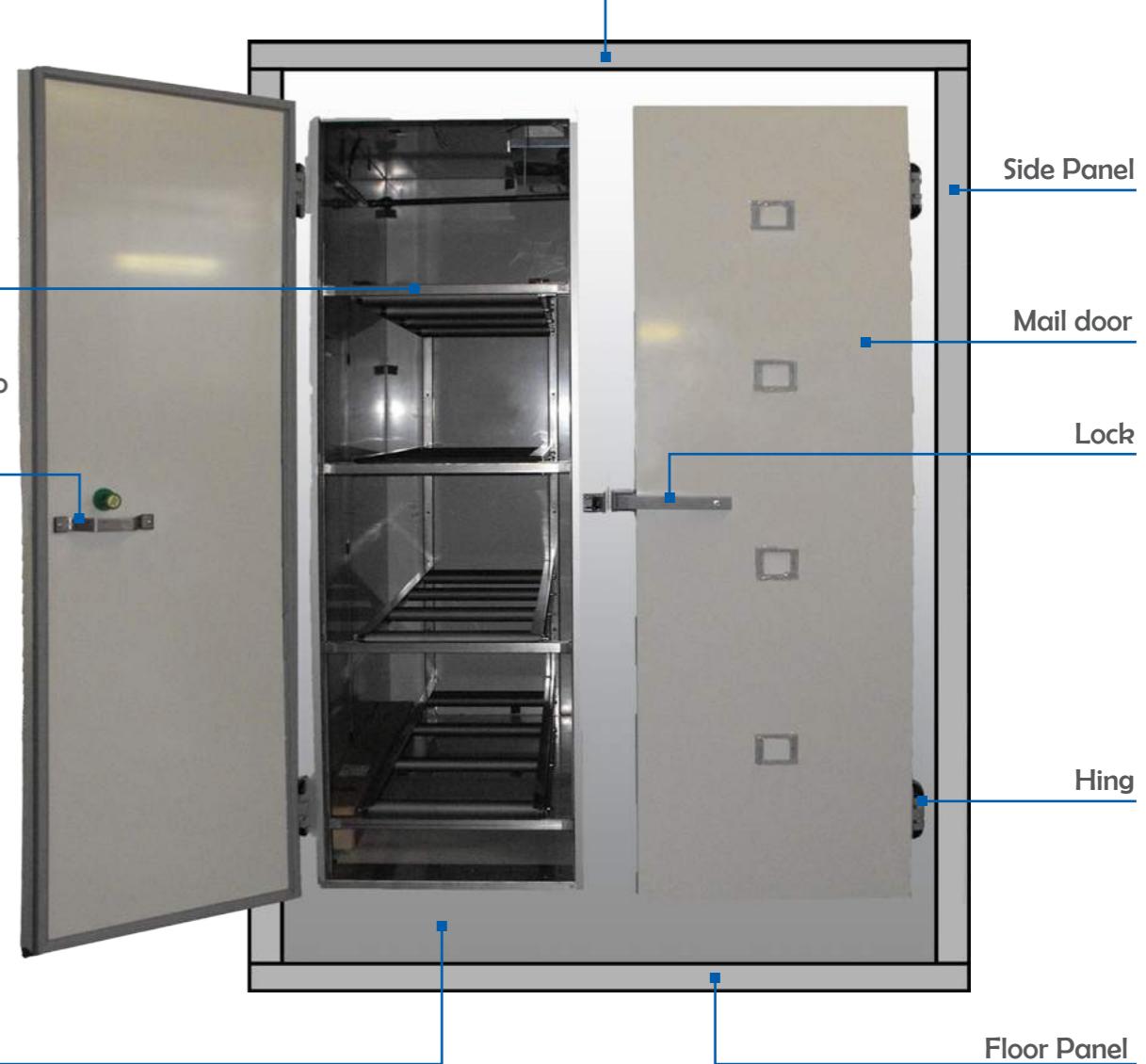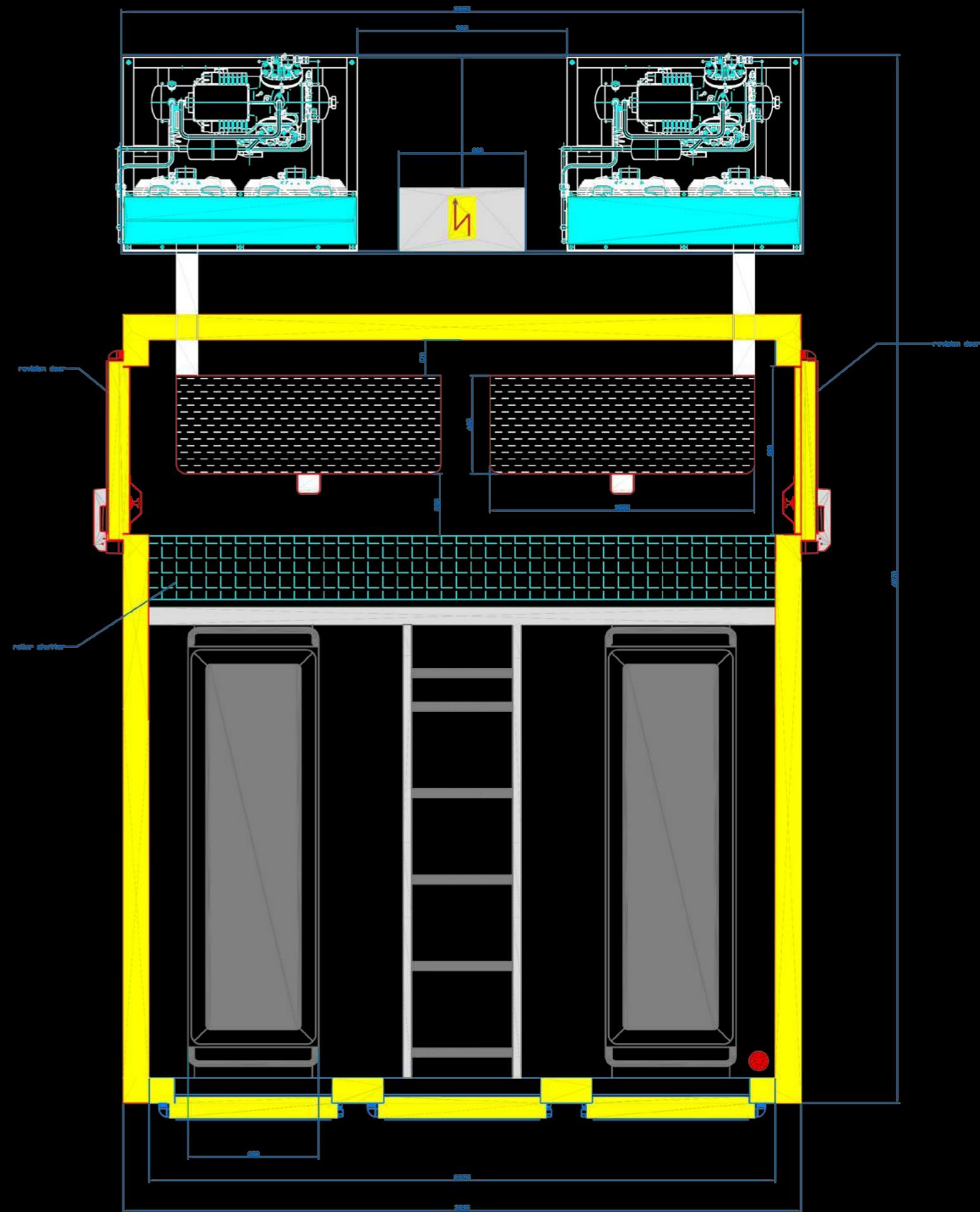
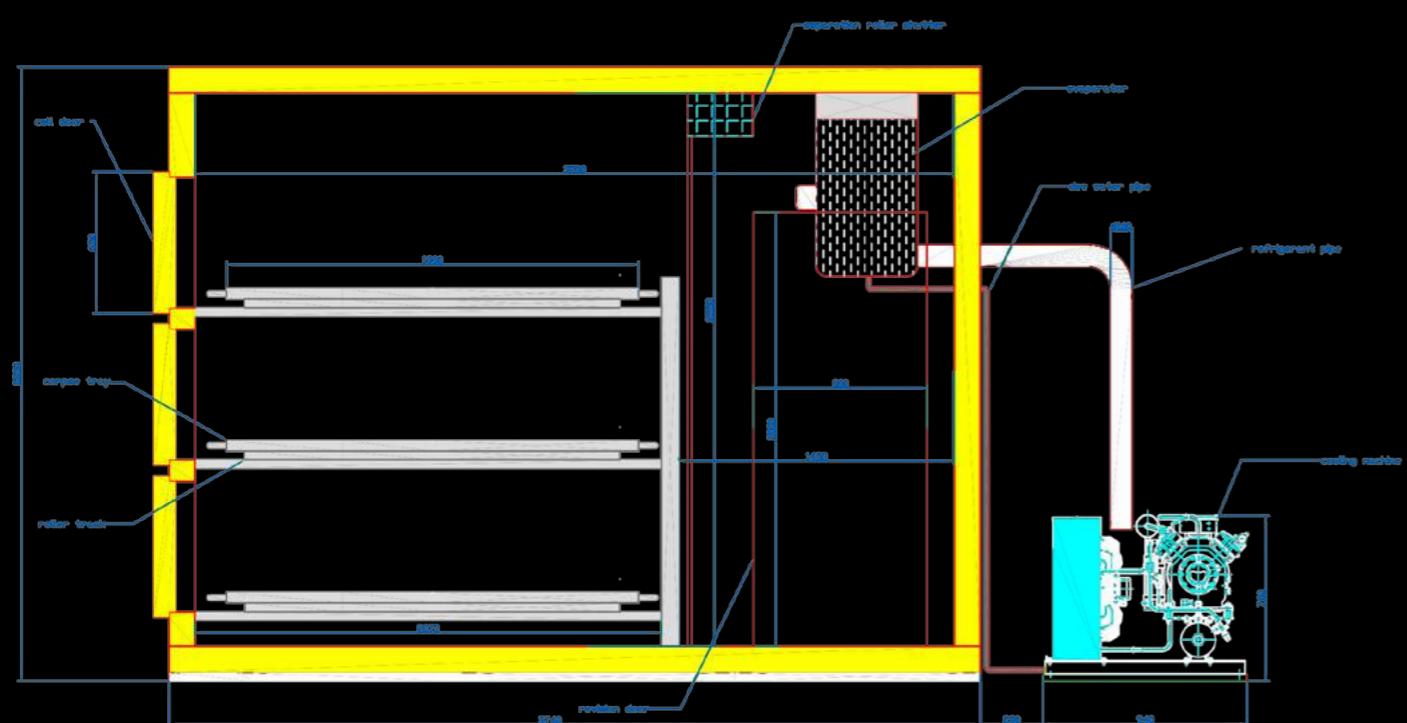
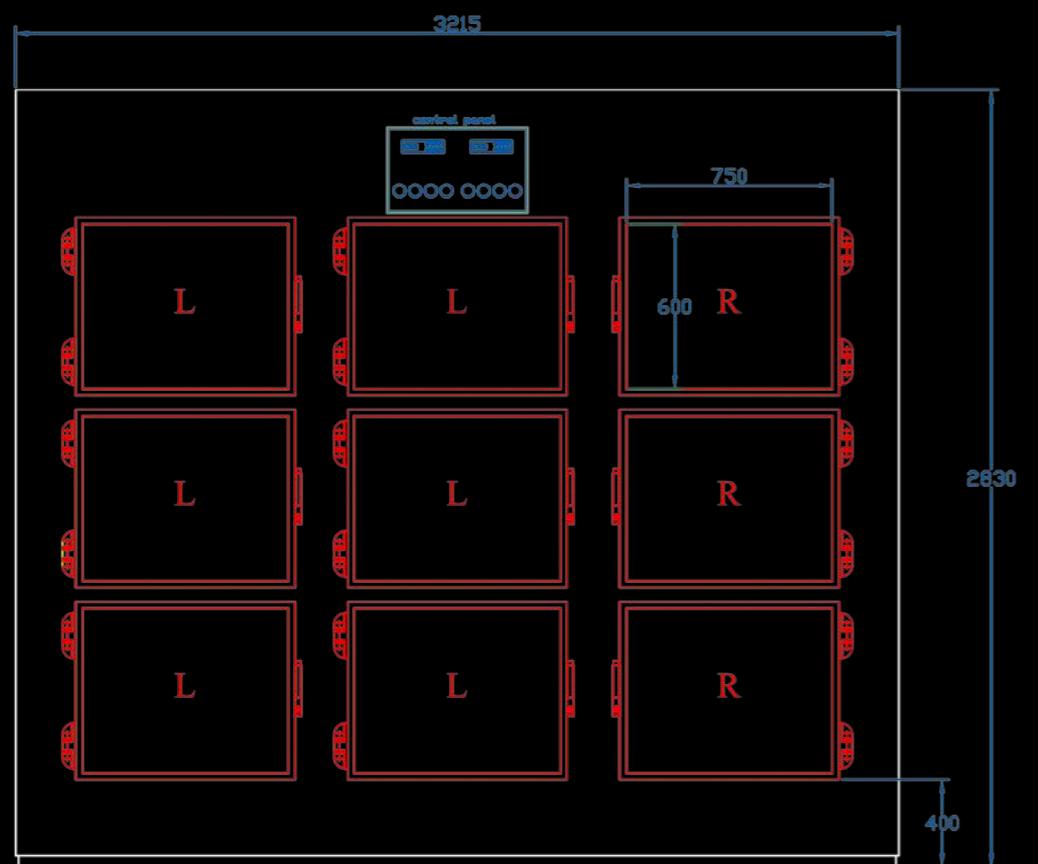
Twelve cells Mortuary Refrigerator

Mortuary refrigerators keep +4° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).

Mortuary freezers keep -20° C preservation temperature at ambient temperature up to 32°C (+ 50°C on request).

Exterior and interior material can be sheet steel lacquered white or stainless steel on request.

Polyurethane foam with low heat transfer coefficient 0.240 W/m² K is used as insulation.



Our mortuary refrigerators have the lowest noise level, are easy to clean and meet hygienic requirements better.

Our ability to satisfy any wish of our customers regarding size, surface materials and way of serving equipment indicates our strength in particular.

We do not produce mass-products - we find answers and solutions.

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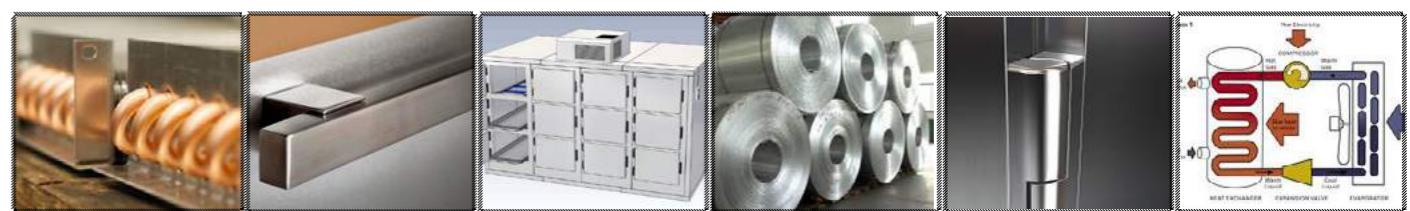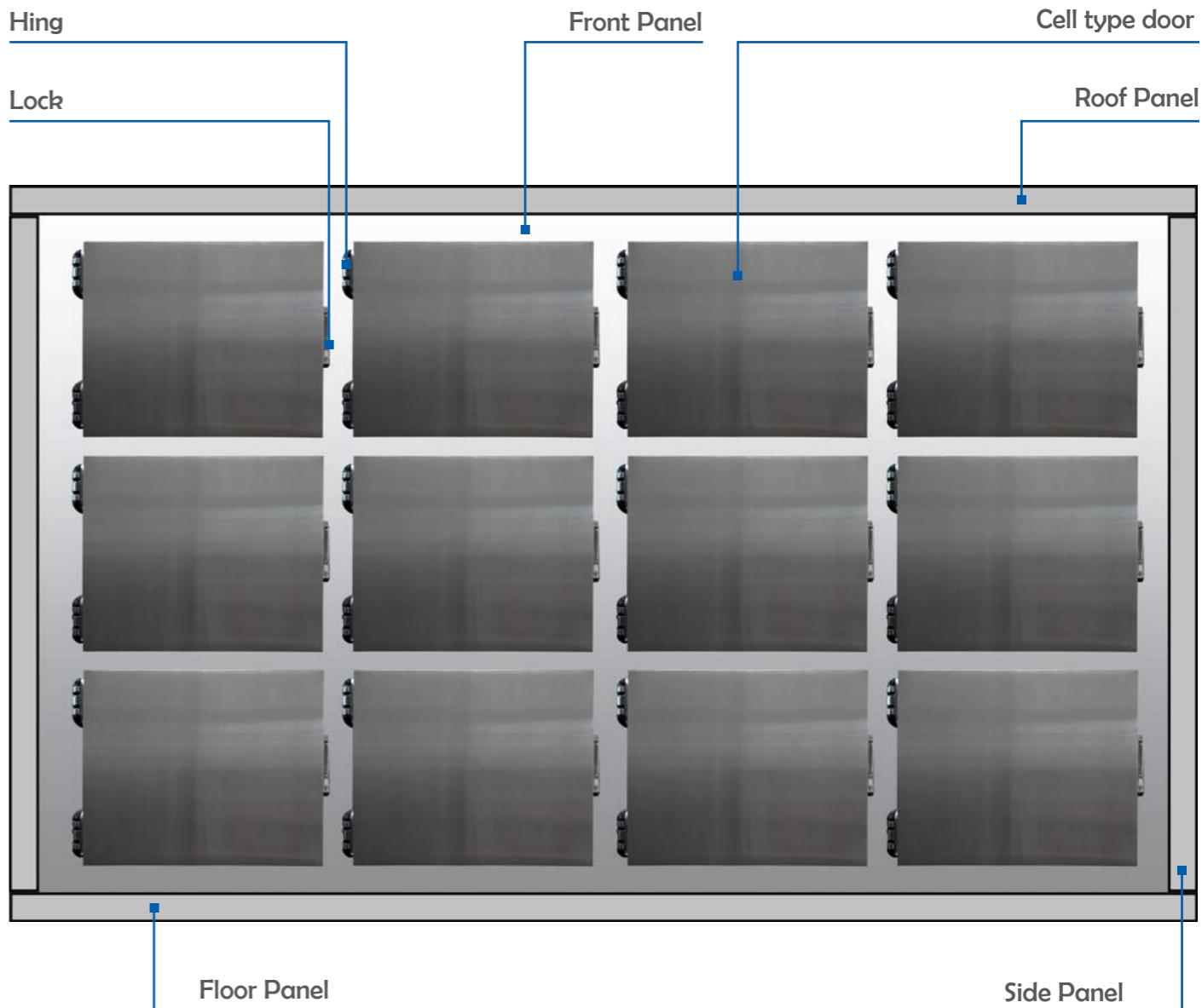Transport cart has rugged, torsion-resistant frame structure of stainless steel, Which contain 5 carrying rollers of corrosion-resistant material and noise absorbing plastic coating is moulded on top of the cart.

Lifting and transport cart has rugged, torsion resistant frame of stainless steel (material 1.4301, ANSI 304) with integrated hydraulic pump for vertical adjustment.

All products are made 100% at the production facilities of Thalheimer in Germany and according to the highest international standard.

Transport carts

Cadaver Cover AH

Cadaver Cover AH

Cadaver Cover AH is made completely of stainless steel type 1.4301 (Aluminum on request) and matches the body tray type LM. The cover's surface is grinded, carrying handles are on both sides.

The cover is absolutely resistant to cleaning agents and disinfectants.

Dimensions AH length/width/height: 1900 x 620 x 340 mm

Weight : approx. 18 kg stainless-steel/ 14 kg aluminum

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Transport Cart LTB

Transport Cart LTB

Transport cart LTB has rugged, torsion-resistant frame structure of stainless steel.

Its 4 steerable solid rubber wheels, 2 of them lockable (diameter 125 mm) with special roller bearings, permanently lubricated.

Stainless steel roller track containing 5 carrying rollers of corrosion resistant material and noise absorbing plastic coating moulded on top Holding device at the roller track secures body tray.

Pneumatic tires are available for transport on uneven ground.

To enable paying last respect in a reverent fashion, the transport cart can be equipped with a black catafalque and/or a stainless steel cover

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Lifting and Transport Cart

Lifting and Transport Cart LTW-MHY

Lifting and transport cart enables to lift and load body trays and coffin plates into mortuary refrigerators by one person.

It has rugged, torsion resistant frame of stainless steel (material 1.4301, ANSI 304) with integrated hydraulic pump for vertical adjustment.

The vertical adjustment is easy done by UP/DOWN lever. Four guided, solid rubber wheels, two lockable, d = 125 mm with special roller bearings, permanently lubricated and maintenance free, ensure absolutely noiseless movement.

A roller track containing 5 rollers made out of corrosion resistant material and plastic coated is mounted on top of the truck. Holding device secures the body tray or coffin plate laying on it. The bow handle of the lift and transport cart can be removed.

The front end is protected with an anti-collision buffer.

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
Go app!

SMARTER EVERY DAY WITH
THE NEW APP

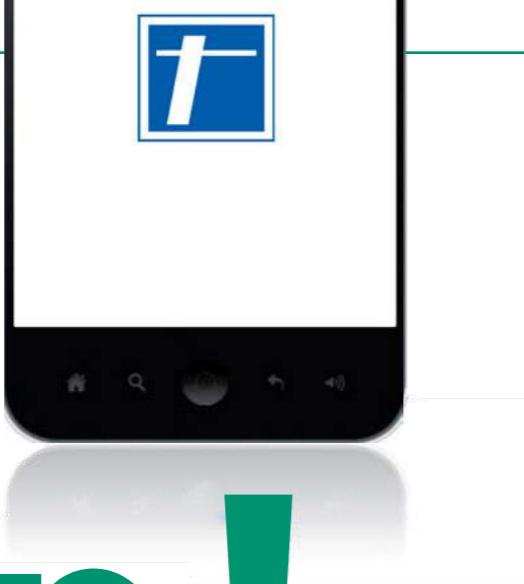With the **new app designed to read the QR-codes** of Thalheimer's website, we have introduced the latest technology into the world of professional Medical Refrigeration system. The app is a first in the industry and allows **product information to be obtained at all times, from wherever you are**. An important tool for both the end user, who can view the instruction manual, and specialized personnel, who can access technical material, interact with the support center and submit orders.

Lifting and Transport Cart

Lifting and Transport Cart LTW-MHY-W (with a scale)

Lifting and transport cart enables to lift and load body trays and coffin plates into mortuary refrigerators as well as weigh a body on the tray by one person.

It has rugged, torsion resistant frame of stainless steel (material 1.4301, ANSI 304) with integrated hydraulic pump for vertical adjustment.

The vertical adjustment is easy done by UP/DOWN lever.

Four guided, solid rubber wheels, two lockable, d = 125 mm with special roller bearings, permanently lubricated and maintenance free, ensure absolutely noiseless movement.

A roller track containing 5 rollers made out of corrosion resistant material and plastic coated is mounted on top of the truck.

Holding device secures the body tray or coffin plate laying on it.

The bow handle of the lift and transport cart can be removed.

The front end is protected with an anti-collision buffer.

Integrated weighting scale enables to weight a body on the tray.

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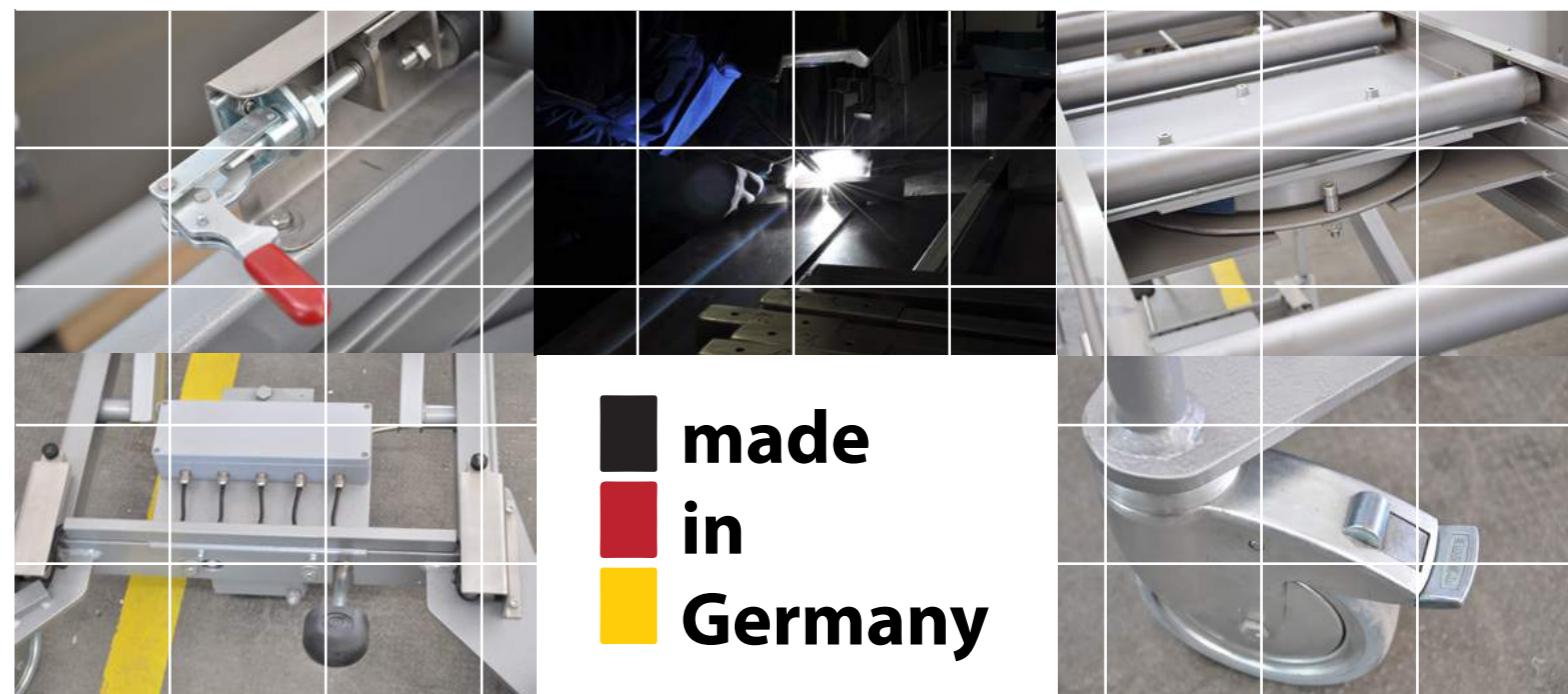
Lifting and Transport Cart

Lifting and Transport Cart LTW-EHY (electro-hydraulic)

The lifting and transport cart enables to lift and load body trays and coffin plates into mortuary refrigerators by one person. It has rugged, torsion resistant frame of stainless steel (material 1.4301, ANSI 304) with integrated electro-hydraulic pump for vertical adjustment. The vertical adjustment is easy done pushing UP/DOWN button. The button is lockable with the key. Four guided, solid rubber wheels, two lockable, d = 125 mm with special roller bearings, permanently lubricated and maintenance free, ensure absolutely noiseless movement. A roller track containing 5 rollers made out of corrosion resistant material and plastic coated is mounted on top of the truck. Holding device secures the body tray or coffin plate laying on it. The bow handle of the lift and transport cart can be removed. The front end is protected with an anti-collision buffer.

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Made in Germany

Dissection Tables

Thalheimer manufacture all Autopsy and dissection tables in-house at our production facilities in Germany. We produce Autopsy and dissection table models SZM1, SZM2, SZM3, SZM4 and KST which all are of different technical specifications to meet all client's requirements. They are height-adjustable between 750 - 1000 mm by electromechanical system pushing a button UP/DOWN and rotatable to the right or to the left through 90° by quick-release handle under the surface.

SZMs is simply fixed at the height 850 mm.

Autopsy and dissection tables SZM 1-3 can be connected to the exhaust system with 1000 m³/h exhaust air capacity.

Autopsy and dissection table SZM4 are made out of heavy gauge stainless steel, DIN 1.4301, their round and

polished surface are embossed and sloped towards the drain. The surface plate has basin with a sieve and a standpipe.

Autopsy tables KST are used in anatomical classes and made of heavy gauge stainless steel DIN 1.4301. A 10 liter container is located beneath the drain outlet for liquid gathering. An air-proof compartment mounted beneath the table surface consists of a stainless steel full extension drawer with a cover and a removable stainless steel perforated tray.

Washing and autopsy table WT is made completely of stainless steel (material 1.4301, AISI 304). The table surface is framed with a rim (60mm at the drain side and 30 mm opposite) and sloped towards the drain. The legs have 4 steerable wheels, 2 are breakable.

Autopsy table KST for anatomical classes

Autopsy table KST for anatomical classes

Autopsy tables KST are used in anatomical classes and made of heavy gauge stainless steel DIN 1.4301. Surface of the table has high sloped frame, diagonally embossed, sloped towards a drain, noise-damped, with splash proof lining on the underside, depth of approx. 15 mm. The table has 4 casters, two of which with brakes, diameter 125 mm.

A 10 liter container is located beneath the drain outlet for liquid gathering. An air-proof compartment mounted beneath the table surface consists of a stainless steel full extension drawer with a cover and a removable stainless steel perforated tray.

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The Seziertisch is completely made of stainless steel (material 1.4301) with a stable frame made of bending stiff stainless steel profiles and 4 lockable stainless steel swivel casters d=125 mm, manufactured. The table top is ca. 15 mm deep, with a slope towards the drain, engraved. The corners of the table surface have a large radius to facilitate cleaning. Under the drain there is a 10 liter container with bayonet closure which is located on a shelf under the table. Under the table top is a closed fabric box for the storage of covers, with a stainless steel pull-out tray and a removable stainless steel perforated tray.

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The autopsy table is entirely made of stainless steel (material 1.4301) with a very resistant frame and 4 pivotable lockable casters d=125 mm. The table top has a thickness of about 15 mm with a slight slope in the direction of evacuation. The angles of the table surface have a large envelope to facilitate cleaning. A 10 liter container with bayonet closure is located under the table on a shelf. Under the table top, a closed fabric box for the storage of covers, with a stainless steel pull-out tray and a removable stainless steel perforated tray.

Dimensions (LxWxH)	Carrying capacity	Weight:
2100 x 800 x 850 mm	200 kg	130 kg
Maße (Lx BxH)	Tragfähigkeit	Eigengewicht
2100 x 800 x 850 mm	200 kg	130 kg
Dimensions L/I/H	Charge admissible	Poids propre
2100 x 800 x 850 mm	200 kg	130 kg

Ventilated dissection table

Ventilated dissection table SZM1V, SZM2V

Ventilated dissection tables (models SZM1V and SZM2V) are made out of heavy gauge stainless steel, DIN 1.4301. The offensive smell is exhausted through polished, perforated and removable surface plate consisting of 3 parts. The surface plate is sloped towards the collecting sieve and framed with a rim. The collecting tank sloped towards the drain is situated under the removable surface plate.

The dissection table has to be connected to the exhaust system with 1000 m³/h exhaust air capacity. The surface plate has basin 500x400x250 mm, knee-operated mixing valve with 3 m long hose shower, 2 plugs 230V/16A.

Model SZM1V is height-adjustable between 700 - 1000 mm by electromechanical system pushing a button UP/DOWN.

Model SZM2V is simply fixed at the height 850 mm.

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Down-Draft Grossing Table

Down-Draft Grossing Table USC-1500

Grossing table is equipped with air down draft system which guarantees laboratory environment free of pollutants while slicing and preparing histological samples. It is also suitable for small dissections. Maximum Allowable Concentration (MAC) of offensive vapors for formalin is under the target values all over the working place. The table is connected to side-mounted exhaust system.

The UCS-1500 model has the following features:

- Working place with large working area
- Entire construction is of stainless steel
- Stable supporting construction with adjustable legs
- Basin 400 x 400 x 200 mm can be mounted left or right
- Stable perforated working plate
- Integrated sprinkle system with adjustable valve
- Formalin sink with sieve
- Tab with detachable hand sprinkler
- Exhaust duct left or right
- PE Cutting table 400 x 500 mm

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Down-Draft Grossing Table

Down-Draft Grossing Table USC-2000B

The grossing table is equipped with air down draft system which guarantees laboratory environment free of pollutants while slicing and preparing histological samples. It is also suitable for small dissections. Maximum Allowable Concentration (MAC) of offensive vapors for formalin is under the target values all over the working place. The table is connected to side-mounted exhaust system.

The UCS-2000B model has the following features:

- Working place with large working area
- Entire construction is of stainless steel
- Stable supporting construction with adjustable legs
- Adjustable height from 800 mm to 1100 mm
- Basin 400 x 400 x 200 mm can be mounted left or right
- Stable perforated removable working plates
- Integrated sprinkle system with adjustable valve
- Formalin sink with sieve
- Tab with detachable hand sprinkler
- Exhaust duct left or right
- Built-in "day light" illumination 5000 K for taking pictures or video without additional flashlights
- Holding device for video camera
- Video camera (transmission possible)
- 230 Volt electric socket outlet
- Holder for scales, documents and Dictaphone (voice recorder)

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Washing and Autopsy Table

Washing and Autopsy Table WT

Washing and autopsy table is made completely of stainless steel (material 1.4301, AISI 304). The table surface is framed with a rim (60mm at the drain side and 30 mm opposite) and sloped towards the drain. The legs have 4 steerable wheels, 2 are breakable equipped with permanently lubricated and maintenance-free special roller bearings.

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Washing and Autopsy Table WT with Installation Column

Installation column INSS has two couplers to fix a washing and dissection table model WT, revision opening at the back side, an organ sink with stand pipe and one-handle water mixer. One extra one-handle water mixer with 3 m long sprinkler hose and sprinkler head is located on the right side of the column. Enclosed drain outlet with a drain pipe for the washing and dissection table is mounted on the front. The whole construction is made out of stainless steel material 1.4301 (ANSI 304).

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www.thalheimer-kuhlung.com

www.thalheimer-kuhlung.com

Please contact us for more information:

Head Office Germany

Thalheimer Kühlung GmbH & Co. KG
Mühlgraben 68, D- 73479, Ellwangen - Jagst
P.O Box 1364 , 73473 Ellwangen - Germany
Tel + 49 (0) 7961 56 77-0
Fax + 49 (0) 7961 56 77-22
info@thalheimer-kuehlung.com

United Arab Emirates

Thalheimer Kuhlung FZCO
Citadel Tower, Office 1008 – 1009
Burj Khalifa district P.O.Box 40668 Dubai-U.A.E
Tel: +971 (4) 422 1898
Fax: +971 (4) 422 1803
thalheimerme@thalheimer-kuehlung.com

Distributor